



Alpha Pro Tech N-95 Particulate Respirator


PFL® (Positive Facial Lock®) Design
with Magic Arch® and Twist Seal®



Donning Instructions 配戴步驟



Step 1.
Grasp nosepiece at top of mask with one hand. Slide finger of other hand behind chin piece.
一手拿著口罩鼻樑固定條，另一手拿著口罩下巴固定條。

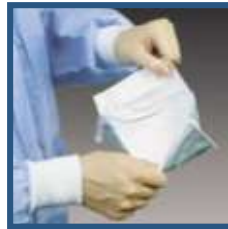

Step 2.
Open mask by pulling down and out on chin piece until mask is fully open. It is **IMPORTANT** that mask is fully open.
兩手把口罩完全打開。



Step 3.
Place chin in pocket created. Stretch bands over back of head, positioning bottom band behind neck below ears and top band near crown of head.
先把口罩套在下巴，戴上臉上，再穿頭帶，一條放在頸，另一條戴在頭部高過耳的位置。



Step 4.
Conform nosepiece to nose and face contours making sure Magic Arch™ is fully extended away from nose and mouth.
把鼻樑固定條跟鼻型貼實，緊貼面型，同時確保口罩中間突出，離開口部。



Step 5.
Holding mask at nosepiece, place one finger inside the mask on the malleable aluminum chin piece, pull down gently so that mask is snug against face.
手固定鼻位，另一手伸食手指入下巴位，手指緩緩向下拉，令口罩緊貼面部。



Step 6.
Pinch any excess material together under chin and fold over or twist.
下巴固定條摺實並扭緊，令口罩鎖定位置。
"Re-adjust the nose piece to make sure it fit well from nose to two edges of face."
"需再調整固定條，確保由鼻至兩邊面頰都緊貼。"



Step 7.
Face Fit Check: To check fit, place both hands along the edges of the respirator and exhale. If air escapes around your nose, adjust the nose piece. If air leaks at the respirator edges, unfold or untwist the chin piece, repeat Steps 5 and 6.
檢查呼氣時口罩邊緣是否有漏氣，如有漏氣重複步驟4或5-6 直至邊緣無漏氣。



Step 8.
PFL Mask correctly fit to face.
PFL N95 口罩已戴妥。

Warning

1. This product does not eliminate the risk of contracting any disease or infection. Change immediately if the respirator comes in contact with blood or fluids, including body fluids. OSHA has not set a permissible exposure level for airborne biohazards.

2. Follow all instructions and warnings on the use of this respirator. Failure to do so will reduce respirator effectiveness, wearer protection and may result in sickness or death. Wear during all times of exposure. For additional information refer to your supervisor and/or see dispenser box.

3. Before use, the wearer must be trained by the employer in proper respirator use in accordance with applicable safety and health standards for the contaminant and exposure level in the assigned work area.

Positive Face Lock, Magic Arch and Twist Seal are registered trademarks of Alpha Pro Tech, Inc.

N-95 Surgical Respirator

AlphaProTech
INCORPORATED

